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USER MANUAL  
Postural aids system in the lying STABILO-BED

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Manufacturer takes responsibility for proper usage of the product only when the product was purchased from qualified REH4MAT Sp. z o.o. employee or at specialized medical store!

REH4MAT Sp. z o.o. reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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REH4MAT Sp. z o.o. Company introduced quality management system ISO 13485 regarding production, sales and service.

**The system is certified by Det Norske Veritas.**

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## MODELS

- Lower extremity abduction pillow **P-SS-01**
- Cylindrical cushion **P-SS-02**
- Abduction block for knees **P-SS-03**
- Anatomic-shaped cushion under head **P-SS-04**
- Pillow between legs **P-SS-05**
- Calf wedge **P-SS-06**
- One piece trunk support **P-SS-08**
- Anti bedsore round cushion **P-SS-09**
- Knee wedge **P-SS-10**
- Lower extremity abduction pillow **P-SS-11**
- Lateral positioning wedge **P-SS-12**
- Ring cushion **P-SS-13**
- Semi-soft bolster pillows **P-SS-14**
- 'V' pillow **P-SS-15**
- Universal big cushion **P-SS-16**
- Semi-rigid bolster pillows for trunk support **P-SS-17**
- Headrest **P-SS-18**
- Neck ring cushion **P-SS-19**
- Upper extremity support with protector **P-SS-20**
- Silicone pillow **P-SS-21**
- Thermoactive support cushion **P-SS-22**
- Pediatric immobilizer **P-SS-23**
- Anti-bedsore donut cushion **P-SS-24**
- Head cushion **P-SS-25**
- Orthopedic neck pillow **P-SS-26**
- Reflux baby wedge **P-SS-27**
- Pediatric lower limbs abduction wedge **P-SS-28**
- Universal positioning cushion **P-SS-29**
- Lower limbs positioning wedge **P-SS-30**
- Universal cushion **P-SS-31**
- Cylindrical cushion with the abduction wedge and side supports **P-SS-32**
- Lower limbs positioning cushion **P-SS-33**
- Lower limbs positioning cushion **P-SS-34**
- Lower limbs positioning cushion **P-SS-35**
- Lower limbs positioning cushion **P-SS-36**
- Upper limbs positioning cushion **P-SS-37**
- Sensory bean bag floor cushion **P-SS-38**
- Trunk positioning cushion **P-SS-39**
- Trunk positioning cushion **P-SS-40**
- Trunk positioning cushion **P-SS-41**
- Trunk positioning cushion **P-SS-42**
- Trunk positioning cushion **P-SS-43**
- Head stabilization cushion **P-SS-44**
- Neck stabilization cushion **P-SS-45**
- Trunk stabilization cushion **P-SS-46**
- Trunk stabilization cushion **P-SS-47**
- Trunk stabilization cushion **P-SS-48**
- Trunk stabilization cushion **P-SS-49**
- Trunk stabilization cushion **P-SS-50**
- Universal limb separator **P-SS-51**
- Anti-bedsore cushion for the lower limbs **P-SS-52**
- Patient transfer belt **P-SS-53**
- Changing position cushion **P-SS-54**
- Trapezoid headrest **P-SS-55**
- Thermoactive elevation arm wedge **K-SS-01**
- Wedge with self-gripping function **K-SS-02**
- Backrest with self-gripping function **K-SS-05**

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## MODELS

- Thermoactive lower limb raising wedge **K-SS-06**
- Thermoactive Braun splint **K-SS-07**
- Shaped wedge **K-SS-08**
- Lower limb raising wedge **K-SS-09**
- Braun splint **K-SS-10**
- Braun splint with foot support **K-SS-11**
- Wedge **K-SS-12**
- Wedge **K-SS-13**
- Wedge **K-SS-14**
- Wedge **K-SS-15**
- Wedge **K-SS-16**
- Wedge **K-SS-17**
- Wedge **K-SS-18**
- Wedge **K-SS-19**
- Wedge **K-SS-20**
- Wedge **K-SS-21**
- Wedge **K-SS-22**
- Wedge **K-SS-23**
- Wedge **K-SS-24**
- Wedge **K-SS-25**
- Wedge **K-SS-26**
- Wedge **K-SS-27**
- Wedge **K-SS-28**
- Wedge **K-SS-29**
- Wedge **K-SS-30**
- Long lower limb separator **K-SS-31**
- Rehabilitation roller **W-SS-01**
- Rehabilitation roller **W-SS-02**
- Rehabilitation roller **W-SS-03**
- Rehabilitation roller **W-SS-04**
- Rehabilitation roller **W-SS-05**
- Rehabilitation roller **W-SS-06**
- Rehabilitation roller **W-SS-07**
- Rehabilitation roller **W-SS-08**
- Rehabilitation roller **W-SS-09**
- Rehabilitation roller **W-SS-10**
- Form **F-SS-01**
- Form **F-SS-02**
- Form **F-SS-03**
- Form **F-SS-04**
- Form **F-SS-05**
- Form **F-SS-06**
- Form **F-SS-07**
- Form **F-SS-08**
- Form **F-SS-09**
- Form **F-SS-10**
- Modular mattress **M-SS**
- Lower extremity abduction pillow **PM-SS-01**
- Cylindrical cushion **PM-SS-02**
- Calf wedge **PM-SS-06**
- Lateral positioning cushion **PM-SS-08**
- Anti-bedsore round cushion **PM-SS-09**
- Knee wedge **PM-SS-10**
- Lateral positioning wedge **PM-SS-12**
- Ring cushion **PM-SS-13**
- Semi-soft bolster pillows **PM-SS-14**
- 'V' pillow **PM-SS-15**
- Semi-rigid bolster pillows for trunk support **PM-SS-17**

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## MODELS

- Headrest **PM-SS-18**
- Pediatric immobilizer **PM-SS-23**
- Orthopedic neck pillow **PM-SS-26**
- Lower limb positioning wedge **PM-SS-30**
- Cylindrical cushion with the abduction wedge and side supports **PM-SS-32**
- Cushion for the lower limbs with abduction wedge **PM-SS-56**
- Thermoactive elevation arm wedge **KM-SS-01**
- Wedge **KM-SS-02**
- Shaped wedge **KM-SS-08**
- Braun splint **KM-SS-10**
- Rehabilitation half-roller **PW-SS-01**
- Rehabilitation half-roller **PW-SS-02**
- Rehabilitation half-roller **PW-SS-03**
- Rehabilitation half-roller **PW-SS-04**
- Rehabilitation half-roller **PW-SS-05**

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## CONTENTS

STABILObed <sup>®</sup> SYSTEM	08
TARGET GROUP	10
INDICATIONS FOR USE	10
CONTRAINDICATIONS FOR USE	10
SPECIFICATION	11
MAINTAINING	12
SAFETY RULES	15
HOW TO FILL THE CUSHION INSERT WITH GRANULES	19
ASSEMBLY OF PILLOWS FOR TUBULAR MATTRESS	21
LABELLING	22

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## STABILObed<sup>®</sup> SYSTEM

**STABILObed<sup>®</sup>** is the modular 1st medical class system that is used for precise and effective positioning the user in the lying position. **STABILObed<sup>®</sup>** system has a wide range of applications. The **STABILObed<sup>®</sup>** system consists of independent elements:

### DIFFERENT SHAPED CUSHIONS

**STABILObed<sup>®</sup>** system is a range of blocks and cushions, used for stabilization of different body parts. They are covered by innovative fabric **PureMed™**. This material is very safe for human and easy to sterilize. Particular components of system can be used for patient immobilization in diagnostic imaging (MRI, CT scan, X-ray). Depending on type, shape and purpose, **STABILObed<sup>®</sup>** cushions are filled with:

**Granulate** – micro-beads with small granulometry. The micro-beads may go smoothly inside the cushion and because of that, it can be anatomic-shaped very easy. What is more, **PureMed™**, with high vapour permeability, allows the air to go outside the cushion and be around the patient's body.

**VISmemo™ foam** - is our innovative memory foam. The foam creates initial shaping under the first pressure. Thereafter, the foam is more soft, because of user's temperature. **VISmemo™** foam is the most active in areas with high bed sores risk. Our innovative foam **VISmemo™** is the best solution for bedsores therapy these days.

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## STABILObed<sup>®</sup> SYSTEM

**Comfort foam** – soft foam with a lot of closed, small air bubbles. It allows to maintain the patient's shape precisely.

**Off-loading foam** – it's rigid technical foam, created in recycling process. The structure is closed what allows to back to initial position even after long-term lying.

**Silicone** – this filling consists of silicone fibers in round shape with 0,5 cm diameter. There are air inside what prevents against the sweating of the user. The cushion filled silicone is very soft and lightweight and can maintain the perfect anatomic shape. It's Pressure-resistant even after long-term lying pressure.

### SELF-ADHESIVE ELASTIC BAND SHEET

The sheet is made of soft, friendly velour. It's base for **STABILObed<sup>®</sup>** cushions and wedges. All of the components of **STABILObed<sup>®</sup>** system are equipped with set of **VelcroFit™** what prevents against its migration on mattress. The sheet is perfect for standard hospital mattresses. It's equipped with elastic band.

### MODULAR MATTRESS WITH STABILObed<sup>®</sup> CUSHIONS ATTACHMENT FUNCTION

The most developed system of stabilization and anti-bedsore prophylaxis. The modular design based on 8 elongated segments filled with granulate, **PureMed™** material and spacer fabric is an invaluable support for the prevention of bed sores and during long-term care. The arrangement of the openings on the whole mattress, corresponding to the different body segments, allows the **STABILObed<sup>®</sup>** elements to be fitted directly onto the mattress and guarantees constant air flow.

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## TARGET GROUP

People requiring stabilization and compensation of deficits resulting from deformation of the osteoarticular system and weakening of postural muscles. People who are immobilized for a long time, at risk of developing pressure ulcers.

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## INDICATIONS FOR USE

Using **STABILObed**<sup>®</sup> postural aids system is particularly appropriate in the following cases:

- As an immobilization in diagnostic imaging (MRI, CT scan, X-ray);
- Orthopedic, neurological, rheumatologic diseases;
- Rehabilitation, palliative care, and positional therapy;
- Long-term care;
- Postoperative immobilization;
- Various body parts relieving;
- Bed sores prevention.

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## CONTRAINDICATIONS FOR USE

Incorrect positioning of a person using the **STABILObed**<sup>®</sup> system may lead to deformation within the joints, and keeping the person lying in one position for a long time significantly increases the risk of pressure ulcers.

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The stabilizing and positioning **STABIObed**<sup>®</sup> system is made of innovative material called **PureMed**<sup>™</sup> in gray color. It has many advantages:

- It is very safe for patients and easy to disinfect, which makes it ideal for use in high risk of infection areas;
- There is a possibility of multiple use, it does not rub off. It is a perfect solution for hospitals or rehabilitation centers because it can be sterilized and used by numerous users;
- It's lightweight, soft, elastic and easy to shape.
- It contains antibacterial components based on inorganic Silver Zeolite;
- It meets all the essential requirements of the medical products of different destination;
- It has **Oeko-Tex**<sup>®</sup> **Standard 100 certification** which is given to products which have exceptionally skin-friendly parameters;
- It is used as cover for hospital mattresses, operating tables and couches;
- It is waterproof and provides high vapour permeability only in one way – the air may migrate from cushion, but the sweat, blood and urine aren't allowed into inside.

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**MAINTAINING****PureMED™ COVER CARE GUIDE**

**PureMed™** should be cleaned according to the laundry guide. Coated materials should be cleaned with the use of neutral pH soap solution (about 7) applied with a soft sponge or a damp cloth. Rub gently and wipe dry to remove the rest of the detergent. In order to keep the original colour and shine level, the procedure should be repeated every month.

Stains should come out if you use 25% solution of ethanol. Rub gently and wipe dry to remove the rest of the detergent. Please, bear in mind that some stains might not be completely removed from the cleaned surface. It depends on several factors like active dyers permanently migrating to the material surface, the material composition and finish, or chemical composition of dirt as well as the time the soiling has remained on the material surface.

You should pay particular attention to the generally available domestic cleansers whose components might react with the soiling and cause an effect contrary to what was originally intended.

Remember that **PureMed™** should never be cleaned with domestic cleansers and cleaning agents that are generally recommended for 'eco-leather'.

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## MAINTAINING

The following table shows the methods and agents approved for disinfection **PureMed™**.

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- **PureMed™** fabric can be disinfected and steam sterilized at 121°C for 20 minutes, minimum 50 cycles;
- The fabric can also be disinfected with hydrogen peroxide, including hydrogen peroxide gas;
- The deterioration of the visual and aesthetic properties (such as wrinklin of the surface) does not affect the fabric's features (both physical and mechanical).

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The product is not covered by the guarantee when:

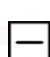
- The damage is caused by high temperature, caustic fluids or fire;
- The permanent discoloration caused by contact with clothing occurs (such as leather, jeans, suede, corduroy, etc.);
- It has traces from a pen, ink, markers, etc.;
- There is a mechanical damage caused by pets or other users;
- The defects are caused by improper maintenance.

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## MAINTAINING

### PureMED™ COVER CARE GUIDE\*

-  Wash at 95°C or less. Gentle process (min 50 cycles).
-  Tumble dry. Medium temperature (drying at standard temperature).
-  Non-Chlorine (do not use compounds that emit free chlorine for bleaching)!
-  Do not iron (avoid contact with heated surfaces, e.g. heater)!
-  Do not dry clean!

### B-SS SELF-ADHESIVE SHEET CARE GUIDE

-  Wash gently at or below 40°C.
-  Do not iron!
-  Non-Chlorine!
-  Do not tumble dry!
-  Dry flat. Keep away from heat sources!

### THE TERRY COVER CARE GUIDE

-  Wash at or below 40°C.
-  Iron inside out, low temperature.
-  Non-Chlorine!
-  Do not dry clean!

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\* Remove the granulate bag before washing! The care guide relates to the PureMed™ cover only.

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## SAFETY RULES

Attention! The product should be used according to the manual's recommendations. If you have any questions, please consult your care professional.

Attention! Before using the product, read this manual carefully. Following the instructions included in the manual will provide product's safety and effectiveness. If you have any questions, please consult your care professional.

Attention! Using of the product not according to the manual may cause serious complications

Attention! Please, keep the manual for future.

Attention! Check the product's condition. If the product is damaged, discontinue use and contact the manufacturer. Only an undamaged product and proper use guarantee its right and safe working.

Attention! Manufacturer is responsible only for hidden defects in the product or mechanical damages occurred in the natural way of using.

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## SAFETY RULES

Attention! Manufacturer is responsible for product's effectiveness and safety only in case of purchasing the product in the authorized seller.

Attention! Pain is always the sign of warning. If the pain maintains or increases, it is advisable to consult your doctor or physical therapist.

Attention! The product should be used according to the recommendations of a physician or physiotherapist.

Attention! Before first using, check the size of the product. Only good size guarantees safety and effectiveness. A wrong size of the product decreases effect of treatment, increases discomfort and wrong position of the user. Proper use of the brace diagnosis by physician orr physician therapist.

Attention! Control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.

Attention! The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occuring, it is necessary to contact a physician.

Attention! In case of varicose veins, Deep Vein Thrombosis (DVT), oedema the using of product is allowed only after contact with physician.

Attention! Before using remove package and accessories like papers, cards or labels.

Attention! Do not use the product against its purpose of use.

Attention! Do not use the product in the wrong size.

Attention! Do not use the product by people and children with mental disorders without guardian control.

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## SAFETY RULES

Attention! Do not use the product in case of damage or shape's deformation.

Attention! Do not attempt to service this product yourself.

Attention! Do not use the accessories which are not recommended by manufacturer.

Attention! Product can be used by many users, but only after disinfection! Follow the PureMed™ washable cover maintenance guide contained in this user manual.

Attention! Before using our products, there is necessary to sterilize them with allowed agents shown in table 2 every time.

Attention! There is recommended to use sheet by single user. In case of using it by more than 1 person, there is necessary to wash it after using, according to maintain rules.

Attention! For proper function, the sheet have to cover the mattress precisely. It cannot be fold. It should be tight and slid under the mattress.

Attention! The service valve should be always closed. You can only open it to adjust the granulate volume.

Attention! Keep our products away from sharp tools and any other items, that can damage STABILObed<sup>®</sup> products.

Attention! In case of mechanical damage one of our product, there is necessary to return it to manufacturer to repair.

Attention! STABILObed<sup>®</sup> system should be kept in dry and clean room. Keeping the other items on our products is forbidden.

Attention! Before using the STABILObed<sup>®</sup> products, always make sure that the place of use (e.g. bed, mattress, seat) is free from sharp edges, metal parts, springs and other elements which may damage the product. The manufacturer is not liable for the mechanical damage of the product!

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## SAFETY RULES

Attention! Product works correctly only when the device is adjusted according to patient's height as well as weight. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

If the symptoms occur (for example skin allergies, intertrigo, bedsores), stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

Attention! Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

The list of all spare parts is available for download on [www.stabilobedsystem.com](http://www.stabilobedsystem.com) after selecting a product.

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## SAFETY RULES

Attention! Over time, the polystyrene granules, which fill the **STABILObed**<sup>®</sup> cushions, may give signs of flattening. As a result, the volume of the cushions will decrease, which is their natural feature. If this situation arises, we recommend replacing or refilling the granules in the cushion. It can be easily done with the use of a special funnel which is in the cushion and a granule bag, which is available in two sizes. The decreasing volume of the cushions is their natural feature and it shall not give rise to a complaint.

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## HOW TO FILL THE CUSHION INSERT WITH GRANULES?

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To supplement the granulaes bag with granules:

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2. Pull out the granulaes bag insert.

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### HOW TO FILL THE CUSHION INSERT WITH GRANULES?

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## ASSEMBLY OF PILLOWS FOR TUBULAR MATTRESS

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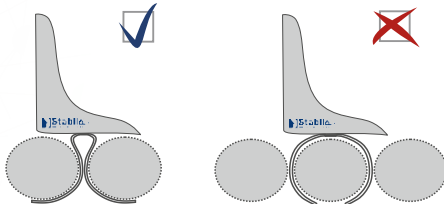
Cushions intended for direct installation on a modular mattress are equipped with self - adhesive straps.

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1. Put the attachment straps through the openings between the mattress segments.

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2. Fasten the Velcro straps as shown in the photo.

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**ATTENTION!** Do not run the straps circumferentially on a single mattress segment. This may result in insufficient stabilization of the cushion and damage to the mattress.

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## LABELLING

 Declaration that the product meets the essential requirements

 Catalog number

 Serial number

 Medical device

 Please read the manual instructions before use

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© Sławomir Wroński 2024