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MAGIC

seat

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FUNCTIONAL POSTURAL SUPPORT POSITIONER

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Manufacturer takes responsibility for proper usage of the product only when the product was purchased from qualified **REH4MAT Sp. z o.o.** employee or at specialized medical store!

REH4MAT Sp. z o.o. reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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REH4MAT Sp. z o.o. Company introduced quality management system ISO 13485 regarding production, sales and service.

The system is certified by Det Norske Veritas.

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CONTENTS

EN

Introduction.....	7
Used materials.....	9
Purpose of user manual.....	11
Indications and contraindications for use.....	11
General characteristic.....	12
Size table.....	13
How to use.....	14
Headrest assembly and height adjustment.....	14
Pelvic belt assembly and adjustment.....	14
Accessories assembly.....	15
Vest.....	15
Thigh abduction belts.....	15
Pelvis stabilizer.....	18
Tray with soft padding and side pockets.....	19
Side pocket.....	20
Soft lower limb pad.....	20
How to fill the insert with granules?.....	20
Lower limbs stabilizing belts.....	22
Side stabilization cushions.....	23
Mobile base with swivel wheels and tilt-angle adjustment.....	23
Adjustable footrest with footers mounted to mobile base.....	25
Height-adjustable base for soft lower limbs pad.....	26
4-points feet stabilizing belts.....	26
The use of device.....	29
Preparation of the user.....	29
Preparation of the device.....	31
Sitting position.....	31
How to use vests.....	33
Safety rules.....	37
Maintenance and cleaning.....	43
Labelling.....	46

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INTRODUCTION

EN

Designed with comfort in mind, the Magic Seat™ is a specialized functional postural support that adapts to the shape of each user's body, providing maximum pressure relief, support and positioning.

Made from high elastic memory foam, the seat provides excellent pressure relief and allows for individual user positioning according to therapeutic indications. Equipped with a pelvic belt and headrest, it ensures an optimal seating position for the user.

In the Magic Seat™ functional postural support positioner, the user's body has minimal contact with the product surface, so the risk of pressure sores is strongly reduced. In addition, different foam combinations can be used to provide the required level of pressure relief and support.

The positioning seat adapts to the needs of each user and makes it easier to maintain a seated posture. The seat is designed to position and support the child's head, trunk and lower limbs in correct alignment. The chest harness provides additional support and protection for the child's torso. It is a professional medical product that facilitates basic activities, i.e. eating, developing correct visual habits and daily care.

The Magic Seat™ has no rigid or hard parts and can therefore be used by children with uncontrolled and involuntary movements, without the risk of impact or injury. Ideal support in the course of diseases of the nervous system manifested by uncoordinated reflexes, such as: hyperkinesia, hypokinesia, athetosis, akathisia, chorea, dyskinesia, dystonia, bismuth encephalopathy, myoclonus, tremor or tics.

Purpose of use:

- cerebral palsy,
- Huntington's disease (HD),
- stroke,
- perinatal hypoxia,
- Wilson's disease,
- phenylketonuria (PKU),
- Central Nervous System Tumors,
- viral infections of the brain,
- epilepsy,
- degenerative diseases,
- spinal muscular atrophy (SMA),
- autism.

ATTENTION!

Before using the product, user is obliged to familiarize himself/herself with user manual. Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.

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EN

POLYURETHANE GRANULATE - SEATREST

The seatrest is filled with fine particles of polyurethane foam, enclosed in a cotton bag. The micro-beads may go smoothly inside the cushion and because of that, it can be anatomic-shaped very easy. What is more, PureMed™, with high vapour permeability, allows the air to go outside the cushion and be around the user's body.

VISmemo™ FOAM - BACKREST

Is our innovative memory foam. The foam creates initial shaping under the first pressure. Thereafter, the foam is more soft, because of user's temperature. VISmemo™ foam is the most active in areas with high bed sores risk.

Our innovative foam VISmemo™ is the best solution for bedsores therapy these days.

POLYSTYRENE GRANULATE - SOFT LOWER LIMB PAD (ACCESSORIES)

The soft lower limb pad is filled with micro-beads with small granulometry. The micro-beads may go smoothly inside the cushion and because of that, it can be anatomic-shaped very easy. What is more, PureMed™, with high vapour permeability, allows the air to go outside the cushion and be around the user's body.

CONSTRUCTION FOAM RICHFOAM® - PRODUCT CONSTRUCTION

RICHFOAM® technical foam by CARPENTER®. It was created through recycling. It has a closed structure, which makes it return to its original shape even after prolonged deformation. It carries the **LGA** safety and quality certification and **Oeko-Tex® Standard 100**.

PureMed™

It is safe for users and easy to disinfect, which makes it ideal for use in high risk of infection areas. There is a possibility of multiple use, it does not rub off. It is a perfect solution for hospitals or rehabilitation centers because it can be sterilized and used by numerous users. It's lightweight, soft, elastic and easy to shape.

It contains antibacterial components based on inorganic Silver Zeolite.

It has **Oeko-Tex® Standard 100 certification** which is given to products which have exceptionally skinfriendly parameters.

It is waterproof and provides high vapour permeability only in one way – the air may migrate from cushion, but the sweat, blood and urine aren't allowed into inside.

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PURPOSE OF USER MANUAL

EN

The instruction manual contains basic information necessary to properly prepare the device before use, use the device, maintain and clean the device.

This user manual should be kept in an easily accessible place. The manual is intended for people who look after disabled users, as well as for doctors and physiotherapists handling the device.

Before using the Magic Seat™ for the first time, the user (or caregiver) is obliged to familiarize himself / herself with the user manual.

INDICATIONS AND CONTRAINDICATIONS FOR USE

These medical devices are intended for people with orthopedic, neurological and cardiological diseases and for people after surgery, with musculoskeletal dysfunctions, who have not acquired the ability to sit independently or who have difficulties in adopting and maintaining the sitting position.

Both for flaccid people with reduced muscle tone and for people with spasticity. The device can also be really useful for people who require long-term relief of one or both lower limbs. It is an irreplaceable aid in the rehabilitation process, both in terms of movement and social integration. The device is great help during everyday activities such as rehabilitation but also during playing, studying or having meals. The selection of the appropriate size of the device depends on medical indications and anatomical parameters, regardless of the age of the user.

The device is recommended for people in the following cases:

- Cerebral palsy – various types and forms;
- Brain and spinal cord injuries with lower limbs paralysis and weakness, balance and motor coordination disorders with different etiology;
- Demyelinating disease (e.g. multiple sclerosis - SM);
- Meningocele;
- Muscles diseases with weakness and paralysis (dystrophy, myasthenia gravis, myositis);
- Genetic disorders;
- Degeneration and neurological diseases;
- Other condition connected with muscles weakness, paralysis and musculoskeletal disorders;
- After head and spinal cord injuries and spine trauma with paralysis and paresis.

Contraindications for sitting:

- Critical condition of the user;
- Paralysis;
- Deep sedation;
- Unstable fractures with specific orthopedic recommendations, including spine fractures;
- Untreated deep vein thrombosis;
- Hemodynamic instability;
- Identified brain dysfunction when sitting upright;
- Severe arthritis of the cervical spine;
- Pulmonary embolism;
- Recent myocardial infarction in the last 24 hours;
- Fresh postoperative wounds or non-healing wounds in the back area, pelvic girdle area, hip joints and thighs;
- Users with mental disorders should not be left in the device without the supervision of a caregiver.

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HEADREST WITH VISmemo™ FOAM
 AND HEIGHT ADJUSTMENT

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SOFT, ANTI - TRAUMATIC
 SEAT STRUCTURE

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BACKREST CUSHION
 WITH VISmemo™ FOAM

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
SEAT CUSHION
 WITH POLYURETHANE GRANULATE

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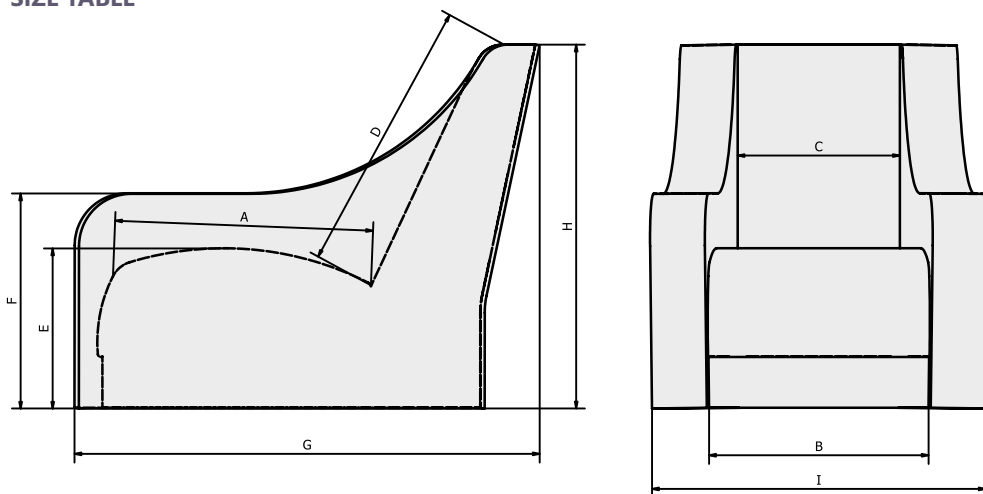The height of the headrest is adjusted by adjusting the length of the headrest mounting straps using the adjusters.

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PELVIC BELT ASSEMBLY AND ADJUSTMENT

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





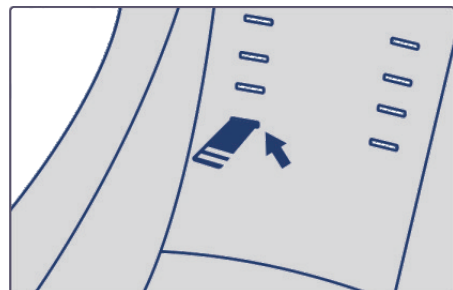
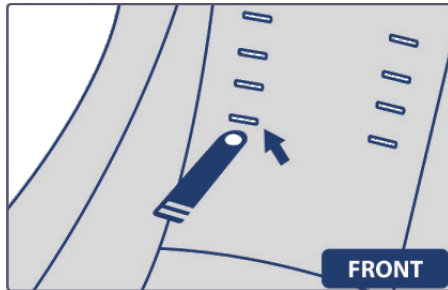
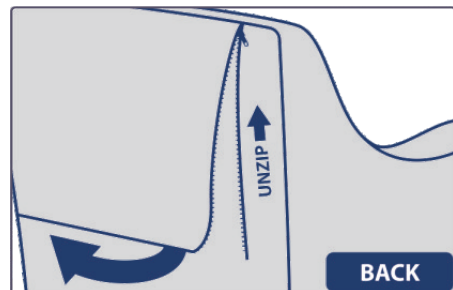
The tool is in a pocket located in the fabric under the backrest flap.

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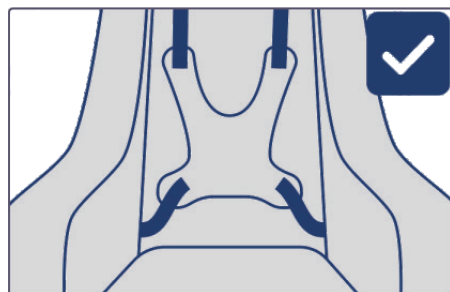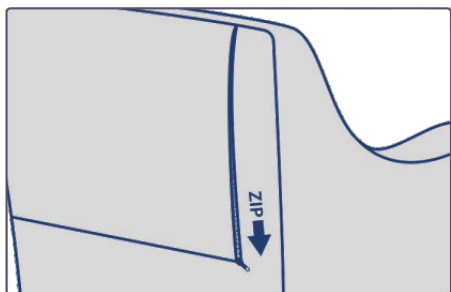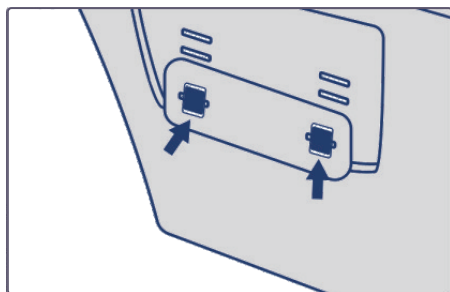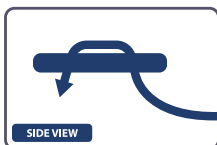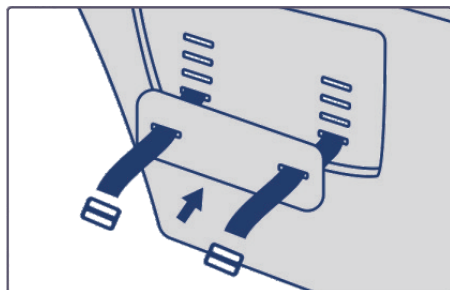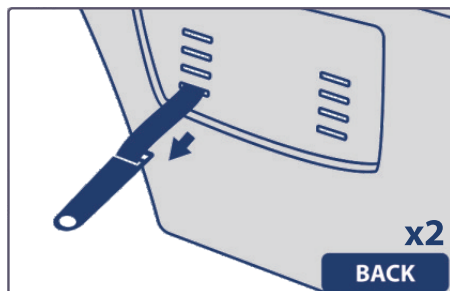
Before using the vest, it is essential to familiarize yourself with the safety rules - see page 33 of the user manual. Improper use and not obeying the safety rules may endanger the user's health and life.

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
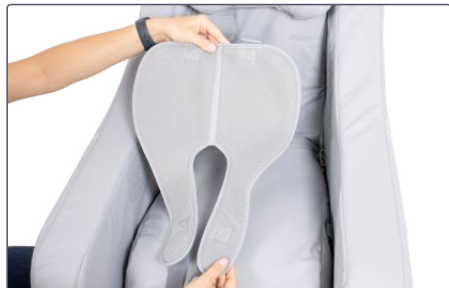
Thigh abduction belts should be mounted so that the AirMesh™ side faces up.

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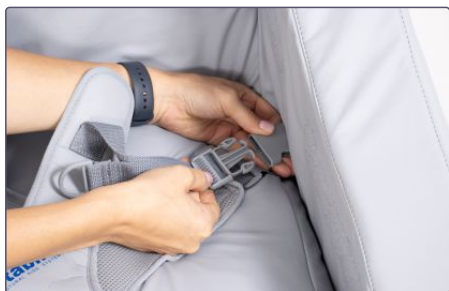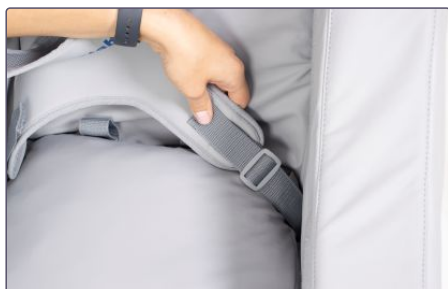Pelvis stabilizer should be mounted so that the AirMesh™ side faces up.

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The tray has a removable inner inside, which has a layer of soft foam. Depending on user needs, the inner can be moved to obtain two different degrees of hardness of the tray.

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Place the funnel with the granulate inside the pocket and pour in the filling.

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After filling, put the protection cap on the granule bag, close the insert and place it in the seatrest.

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The attachment pieces for mounting the side stabilizing cushions are located internally, on both sides of the seat.

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If using the seat with a mobile base, be sure to keep the wheels locked when planting the user on the seat!

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Before using the seat, make sure that the seat tilt mechanism is locked on both sides, at the same height. Release the base tilt lock by gently raising and lowering it.

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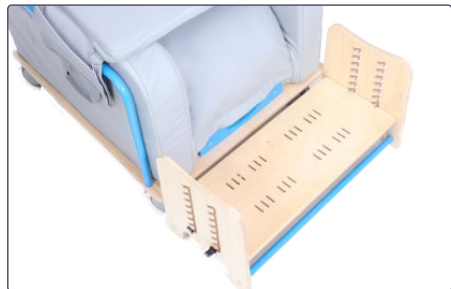
The footrest is adjustable in height and tilt angle.

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
4-PUNKTOWE PASKI STÓP 4-POINTS FEET STABILIZING BELTS

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

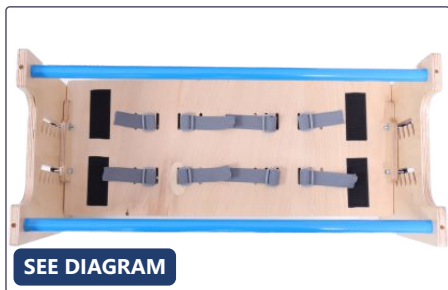
Directly on the base can be mounted 4-point feet stabilizing belts or foot and ankle stabilizing sandals.

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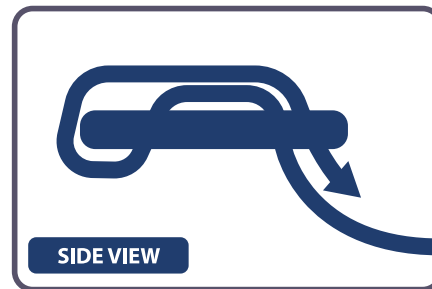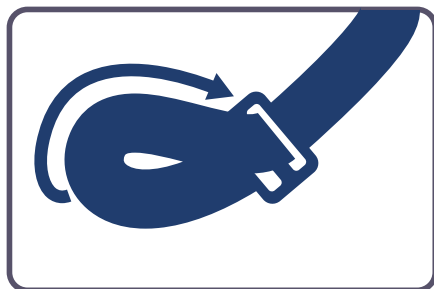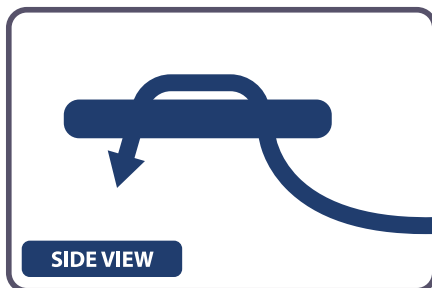The 4-point feet stabilizing belts are attached by putting the straps through the holes in the footrest and the adjuster. Diagram below.

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The 4-point feet stabilizing belts - II method of assembly.

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Magic Seat™ seat has to be used with compliance to guidelines included in this user manual.
Do not forget to prepare both product as well as user before using Magic Seat™.

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Before using the product, it is essential to make sure that:

- user's outfit is light and comfortable sporty clothes are advisable,
- due to hygienic reason and to avoid abrasion it is advisable for the user to always wear clothes when using the device, skin must not be naked especially in places of regular contact with upholstery,
- drains, bags, catheters etc. if applied, have to be secured to function properly and not to disturb during using the product,
- if it is necessary for a user to wear orthopedic supply (collar, corset, orthosis etc.) if it is recommended by doctor.

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PREPARATION OF THE DEVICE

Before use it is necessary:

- place the device on even ground,
- make sure that all parts are serviceable and correctly mounted,
- when using a mobile base, block brakes,
- adjust the device to user's measurements,
- undo pelvic belt so as to place easily the user in the device,
- place user in the device,
- fasten all elements which stabilize user,
- mount additional equipment (if applicable).

ATTENTION!

Before the product is used, it is highly necessary to make sure that all belts as well as safety equipment are correctly fastened as well as adjusted.

Equipment works correctly only when the device is adjusted according to user's measurements. Remember to take all these measurements into account while purchasing the product.

Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force user to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

SITTING POSITION

The first use of the seat usually takes place at a rehabilitation, neurology, neurosurgery or orthopedics center. It is especially important to correctly adjust each element of the device to the user regarding their condition and taking into account existing contractures and deformations. The first use of the device at home should be done with the help of qualified staff (physiotherapist, health nurse, properly prepared family members) after having read the instruction manual.

ATTENTION!

Incorrect adjustment can lead to such side effects as fainting, body deformities etc.

If the abovementioned symptoms occur, stop using the device immediately, take the user out of the device and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

The decision to use the device must always be made by the attending physician.

Adjusting the device to the user should only be done by people who have been properly trained and have read the instruction manual.

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EN

HOW TO USE VESTS

The chest harness provides posture support for users sitting in positioning chairs or other rehabilitation seats. When properly used, the chest harness allows the user to adopt the correct sitting position for a longer period of time. The design and shape of the chest harness make it possible for the user to be free to move their upper limbs within the constraints of the user.

If the chest harness is not used properly, it can pose a serious threat to the health and life of the user!

Possible threats incident to the use of the chest harness:

- 1) If the lower chest harness straps are fastened too loosely, there is a risk that the chest harness will move upwards to the user's neck, making it difficult to breathe - always check the tension of the straps and set them in such a way that the lower edge of the chest harness is located near the lower abdomen of the user.
- 2) In a situation when the user slips off the seat, the upper edge of the chest harness covers the user's neck and blocks the trachea, making it difficult to breathe - it is therefore necessary to ALWAYS use the accessories stabilizing the user's pelvis, such as a pelvic belt or pelvis stabilizer.
- 3) There is a risk that the upper straps of the chest harness will press the user's neck, blocking the trachea and making it difficult to breathe. This might happen if the user's body is improperly secured in the seat or stroller - it is then necessary to use the optional accessories of these devices and stabilize the user's body using the lateral supports.
- 4) The chest harness is fastened too tightly to the backrest and as a result the user cannot breathe easily - in this case the chest harness should be loosened in such a way that it is possible for the caregiver to insert a straightened hand between the chest harness and the user's body.

Instructions for the proper use of the chest harness:

- 1) Make sure that the user is sitting in the best possible position:
 - in the middle of the seat,
 - deeply in the seat in such a way that the user's lumbar spine rests against the backrest.
- 2) Secure the user's pelvis with, for example, a pelvic belt or pelvis stabilizer.
- 3) Make sure that the side supports of the device (if used), are set in the correct position. It is required that the users with the poor trunk control use the side supports. The side supports limit the lateral movements of the user's body, which could lead to the compression of the chest harness straps on the neck of the user.
- 4) Secure the chest harness:
 - fasten and adjust the lower straps of the chest harness, so that the lower edge of the chest harness is located near the lower abdomen of the user,
 - fasten the shoulder straps of the chest harness,
 - check if the buckles of the chest harness do not open under pressure.
- 5) Check the user's position:
 - make sure that the chest harness cannot move by itself towards the user's neck,
 - make sure that you can insert a straightened hand between the chest harness and the user's body.

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EN

HOW TO USE VESTS

To prevent injuries and to minimize the risks associated with the use of the chest harness, the following rules should be followed:

- 1) The position of the chest harness **MUST** be checked each time after putting the user in the seat.
- 2) Special attention should be paid to protecting the user from falling forward, slipping off the seat or lifting the pelvis. The chest harness **MUST** be used with the accessories which will secure the user's pelvis at all times (e.g. the pelvic belt). The use of the chest harness cannot be considered separately but as a part of a general posture support system including supports such as pelvic belts, foot support, torso stabilization, thigh abduction belts.
- 3) Special attention should be paid to the use of stabilization elements. If the user has poor trunk control, the side supports will limit the lateral movements, which could lead to the compression of the chest harness straps on the neck of the user.
- 4) The mounting of the chest harness and adjusting it to the user can only be carried out by a suitably trained person, who is familiar with the instructions for the use of the chest harness and the rules of its safe use.
- 5) Do not make any repairs or modifications to the chest harness on your own. In order to repair or replace any part of the chest harness, please contact the manufacturer.
- 6) Do not use a damaged or incomplete chest harness, as this poses a serious threat to the health and life of the user!
- 7) Do not leave the user in the device unattended!
- 8) The correctness of the use of the chest harness should be particularly controlled in accordance with the recommendations in case of children, users with impaired memory or mentally ill.
- 9) Make sure that all people taking care of the user are familiar with the rules of the safe use of the chest harness.

How to care for the chest harness:

- 1) The chest harness straps are not frayed or worn. If so, have them exchanged for new ones.
- 2) The chest harness straps are not attached to the wheels of the device.
- 3) The buckles are not damaged. If so, have them exchanged for new ones.
- 4) There have been no changes in the user's height or weight from the moment the chest harness was adjusted to the user.
- 5) The shoulder straps of the chest harness are adjusted so that they are not too loose or too tight.
- 6) The chest harness should be cleaned regularly with the use of warm soapy water and dried out of the direct sunlight and other sources of heat. The chest harness must always be dry before usage.

If you have any doubts or questions about the use of the chest harness, please contact your doctor or physiotherapist.

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SAFETY RULES**EN****ATTENTION!**

Magic Seat™ has to be used with compliance to guidelines included in this user manual. Do not forget to prepare both product as well as user.

It is highly important to consult either doctor or physiotherapist in order to make right decision concerning the choice of Magic Seat™ and its size. It is also essential to familiarize with the guidelines described in user manual before using the device.

- 1) Before use it is necessary to:
 - make sure that there are no damages or different faults which could endanger user's life,
 - check if all parts are working and are correctly mounted and, moreover, check if they are not broken,
 - check if all glued, sewed or tapped parts are tightened,
 - check upholstery for signs of damage,,
 - check if buckles are operating correctly,
 - adjust the stabilizing elements e.g. pelvic belt, headrest, according to user's measurement,
 - make sure that all movable parts and knobs for adjustments are stable and locked,
 - secure user with the aid of elements. It is highly important to secure user because it safeguards him/her from falling out,
 - if abduction contracture occurs, it is recommended to use abduction block in order to prevent ulcers between knees.
- 2) It is forbidden to use product for different purposes that it is intended to.
- 3) It is forbidden to use product when it is damaged or any part is missing.
- 4) It is forbidden to leave the user unattended.
- 5) Product should be used only when there is someone who is trained and knowledgeable.
- 6) It is forbidden to leave product and the user on unstable ground unattended, even if the brake is engaged.
- 7) It is forbidden to either go up or down the stairs; it doesn't matter if the user is placed in the device or not.
- 8) It is important to engage brake during getting into and getting out the device.
- 9) It's important to make sure that user doesn't stand on footer during either getting into or getting out the device.
- 10) Make sure that all limbs are not close to spokes or other adjustable elements.
- 11) Please pay your attention when approaching to sources of fire or sources of high temperature during using the device. Moreover, device shouldn't be stored in rooms with high humidity.
- 12) Secure device during transportation in order to avoid any possible damages. Manufacturer doesn't take responsibility for damages which appeared due to improper preparation of the device during transportation.
- 13) Before using the device, remove all paper and plastic packaging. The packaging is not a toy! Keep away from children and animals.

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SAFETY RULES**EN****ATTENTION!**

Incorrect product adjustment can lead to such side effects as fainting, body deformities, joint overload, etc. If the abovementioned symptoms occur, stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

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It is forbidden to place the user in the device while being in car, bus or train.

This device can be only used inside the buildings.

Do not use a product that has any signs of damage, tearing or wear. Only an undamaged product and proper use guarantee its right and safe working. If the product is damaged, discontinue use and contact the manufacturer.

Under no circumstances should an unqualified person repair or alter the product and its construction, as the health and safety of the user depend on it as well as it will lead to the loss of warranty.

The product should be used according to the manual's recommendations. If you have any questions, please consult your care professional.

Before using the product, read this manual carefully. Following the instructions included in the manual will provide product's safety and effectiveness. If you have any questions, please consult your care professional.

Using of the product not according to the manual may cause serious complications.

Please, keep the manual for future.

Check the product's condition. If the product is damaged, discontinue use and contact the manufacturer. Only an undamaged product and proper use guarantee its right and safe working.

Manufacturer is responsible only for hidden defects in the product or mechanical damages occurred in the natural way of using.

Manufacturer is responsible for product's effectiveness and safety only in case of purchasing the product in the authorized seller.

Pain is always the sign of warning. If the pain maintains or increases, it is advisable to consult your doctor or physical therapist.

The product should be used according to the recommendations of a physician or physiotherapist.

Control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.

The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occurring, it is necessary to contact a physician.

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SAFETY RULES**EN****ATTENTION!**

Before using remove package and accessories like papers, cards or labels.

Do not use the product against its purpose of use.

Do not use the product in the wrong size.

Do not use the product by people and children with mental disorders without guardian control.

Do not use the product in case of damage or shape's deformation.

Do not attempt to service this product yourself.

Do not use the accessories which are not recommended by manufacturer.

Product can be used by many users, but only after disinfection! Follow the maintenance guide contained in this user manual.

Before using our products, there is necessary to sterilize them with allowed agents shown in table 2 every time.

The service valve should be always closed. You can only open it to adjust the granulate volume.

Keep our products away from sharp tools and any other items, that can damage product.

In case of mechanical damage one of our product, there is necessary to return it to manufacturer to repair.

Product should be kept in dry and clean room. Keeping the other items on our product is forbidden.

Before using the product, always make sure that the place of use is free from sharp edges, metal parts, springs and other elements which may damage the product. The manufacturer is not liable for the mechanical damage of the product.

Product works correctly only when the device is adjusted according to patient's height as well as weight. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

If the symptoms occur (for example skin allergies, intertrigo, bedsores), stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

Over time, the polystyrene granules may give the signs of flattening. As a result, the volume of the cushions will decrease, which is their natural feature. If this situation arises, we recommend replacing or refilling the granules in the cushion. It can be easily done with the use of a special funnel which is in the cushion and a granule bag, which is available in two sizes. The decreasing volume of the cushions is their natural feature and it shall not give rise to a complaint.

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MAINTENANCE AND CLEANING

EN

Technical specification PureMed™	
Weight:	200 ± 15 [g/m ²]
Type of PU coating:	100 [%]
PES backing type:	100 [%]
Flammability:	Flame retardant
Tensile strength along:	≥ 20 [daN/5cm]
Tensile strength across:	≥ 10 [daN/5cm]
Waterproofness:	≥ 2000 [mm water gauge]
Water vapour permeability:	≥ 300 [g/m ² /24h]

TAB.1

Magic Seat™ is made of innovative material called PureMed™ in gray color.

- It is very safe for users and easy to disinfect, which makes it ideal for use in high risk of infection areas.
- There is a possibility of multiple use, it does not rub off. It is a perfect solution for hospitals or rehabilitation centers because it can be sterilized and used by numerous users.
- It's lightweight, soft, elastic and easy to shape.
- It contains antibacterial components based on inorganic Silver Zeolite.
- It meets all the essential requirements of the medical products of different destination.
- It has **Oeko-Tex® Standard 100 certification** which is given to products which have exceptionally skinfriendly parameters.
- It is waterproof and provides high vapour permeability only in one way – the air may migrate from cushion, but the sweat, blood and urine aren't allowed into inside.

PureMed™ cover care guide

PureMed™ should be cleaned according to the laundry guide. Coated materials should be cleaned with the use of neutral pH soap solution (about 7) applied with a soft sponge or a damp cloth. Rub gently and wipe dry to remove the rest of the detergent. In order to keep the original colour and shine level, the procedure should be repeated every month.

Stains should come out if you use 25% solution of ethanol. Rub gently and wipe dry to remove the rest of the detergent. Please, bear in mind that some stains might not be completely removed from the cleaned surface. It depends on several factors like active dyes permanently migrating to the material surface, the material composition and finish, or chemical composition of dirt as well as the time the soiling has remained on the material surface.

You should pay particular attention to the generally available domestic cleansers whose components might react with the soiling and cause an effect contrary to what was originally intended.

Remember that PureMed™ should never be cleaned with domestic cleansers and cleaning agents that are generally recommended for 'eco-leather'.

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TAB.2

PureMed™ fabric can be disinfected and steam sterilized at 121°C for 20 minutes, minimum 50 cycles. The fabric can also be disinfected with hydrogen peroxide, including hydrogen peroxide gas. The deterioration of the visual and aesthetic properties (such as wrinklin of the surface) does not affect the fabric's features (both physical and mechanical).

 INFORMATION!

The product is not covered by the guarantee when:

- the damage is caused by high temperature, caustic fluids or fire,
- the permanent discoloration caused by contact with clothing occurs (such as leather, jeans, suede, corduroy, etc.),
- it has traces from a pen, ink, markers, etc.,
- there is a mechanical damage caused by pets or other users,
- the defects are caused by improper maintenance.

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© Sławomir Wroński